




Paediatric Tube Guide



EDTA



**SST &
Amber**

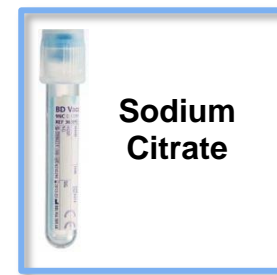

**Sodium
Citrate**



**Lithium
Heparin**

TEST		Minimum Volume
Bilirubin	Amber	0.5 mL
Biochemistry (Full)	SST	2.0 mL
Blood Group	EDTA	0.5 mL
Chromosomes	Heparin	1.0 mL
CK	SST	0.5 mL
Coags (INR, APTT)	Citrate	1.8 mL
Coombs	EDTA	0.5 mL
ESR	EDTA	0.5 mL
EUC	SST	1.0 mL
FBC (+ Rectics)	EDTA	0.5 mL
Iron Studies	SST	0.5 mL
Fragile X	EDTA	0.5 mL
G6PD	EDTA	0.5 mL

TEST		Minimum Volume
Gentamicin	SST	0.5 mL
Glucose (BSL)	SST	0.5 mL
Ig (Immunoglobulins)	SST	0.5 mL
Lead (Pb)	EDTA	0.5 mL
Lithium	SST	1.0 mL
LFT	SST	1.0 mL
Magnesium	SST	0.5 mL
RAST	SST	0.5 mL
Rubella	SST	0.5 mL
TFT (FT4, TSH)	SST	0.75 mL
TSH	SST	1.0 mL
TTG (Tissue Transglutaminase)	SST	1.0 mL
Vitamin D	SST	0.6 mL

Maximum amount of blood to be collected from children under the age of 14	
WEIGHT (kg)	Maximum (mL)
2.7 – 3.6	2.5
3.6 – 4.5	3.5
4.5 – 6.8	5
6.8 – 18.2	10
18.2 – 27.3	20
27.3 – 29.5	25
29.5 – 45.5	30

If it is not possible to obtain the minimum amounts, please phone DSC for advice.

Doctors' Service Centre (DSC) 02 6285 9803

Labelling must include surname, first name, date of birth, and date and time of collection.

For any other tests, please refer to the Test Collection Manual on our website (www.capitalpath.com.au) or contact the laboratory on 02 6285 9803

Issued November 2016